**LAPAROX®**

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**Dioxyde de carbone médical**

**Anidride carbonica medicale**

**Medical Carbon Dioxide**

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# ENGLISH

## Instructions for use – Information for the user

Read the entire instructions for use carefully because they contain important information. The instructions for use are an integral part of the medical device and must always be available to the user, e.g., the physician. Before working with the medical device, you must know and understand its function and handling.

The medical device LAPAROX® consists of compressed liquefied Carbon Dioxide (CO<sub>2</sub>) gas. The liquefied gas is filled in dedicated steel or aluminium cylinders as the primary packaging, using a valve as the closure system.

Please note that a number of the points listed in this Instruction for Use may also pose a safety risk to the physician / user of LAPAROX®. Therefore, it is particularly important that the use of LAPAROX® must always be carried out by staff who have been trained with regard to the hazards associated with LAPAROX®.

For detailed information, see also chapters 5, 6, and 7.

## **This instruction for use include:**

1. Intended Purpose	79
2. Intended Clinical Benefits	79
3. Intended use environment	79
4. Indication, Dosage, Method and Duration of Use	80
5. Contraindications, Warnings, Precautions, Side-Effects	82
6. Precautions and Warnings for Transport and Storage	86
7. Safe Withdrawal	87
8. Cleaning and Disinfection	93
9. Maintenance, Servicing and Disposal	95
10. Further Information	96
11. Labelling	98

### **1. INTENDED PURPOSE**

CO<sub>2</sub> is intended to be used in combination with other medical devices for the following purposes:

- As insufflation gas for surgical and diagnostic endoscopic procedures through body orifices and artificial orifices

### **2. INTENDED CLINICAL BENEFITS**

Medical CO<sub>2</sub> is a well-established insufflation medium which has a proven history to be safe and effective in establishing an adequate visible field in all medical indications described in chapter 4.1.

### **3. INTENDED USE ENVIRONMENT**

Health care facilities.

No specific ambient conditions apply, other than those listed under section 6 «Precautions for Transport / Storage».

## **4. INDICATION, DOSAGE, METHOD, AND DURATION OF USE**

### **4.1 Medical indication**

CO<sub>2</sub> gaseous (cylinder without dip tube) for insufflation to enhance visibility in the following medical procedures:

- Laparoscopy
- Diagnostic hysteroscopy
- Endoscopic vein harvesting in coronary artery bypass operations
- Gastrointestinal endoscopic procedures
- Other endoscopic applications

Insufflation is a well-established necessary step for endoscopic procedures. The selection of the insufflation medium -whether CO<sub>2</sub>, an alternative gas or a liquid - is determined by the procedural requirements and the clinical judgement of the healthcare professional, taking into account patient condition, procedural objectives, and available equipment. Consequently, specific endoscopic applications cannot be exhaustively defined, as the choice of medium is made on a case-by-case basis in accordance with established medical guidelines.

### **4.2 Intended patient population**

There are no known limitations to the patient population. Potential limited use in patients with certain clinical predisposition as described in the «Contraindication» and «Precaution» sections.

### **4.3 Dosage and Duration of Use**

The amount, speed and duration of insufflation depend on the medical indication and are to be determined individually by the treating physicians.

Repeated applications depend on the amount of applied CO<sub>2</sub> in the previous treatment, duration of insufflation and on the individual patient condition. This must be considered individually by the treating physicians.

The optimal pressure for the operating space is the lowest pressure at which the physician can perform the safest surgery without compromising the best outcome. A pressure-controlled insufflation system should be used, applying the lowest possible pressure.

### **4.4 Intended users**

#### **Medical personnel (e.g., physicians)**

Medical intervention may only be performed by physicians who have experience in the intended procedure and are trained in the handling of CO<sub>2</sub>.

#### **Personnel for logistics of the cylinders (e.g., technical personnel, nurses)**

No demographic or physical restrictions when working according to the Linde Gas Schweiz AG safety data sheet, which means that users should be trained in the handling of CO<sub>2</sub>.

## 4.5 Method and application

Insufflation should only be performed by physicians who have experience in the intended procedure. It should be performed with a state-of-the-art insufflation system. The insufflation system should be flooded with CO<sub>2</sub> to reduce the risk of air embolism at the start of insufflation. Ensure that the carbon dioxide is sufficiently preheated and humidified, if required.

## 5. CONTRAINDICATIONS, WARNINGS, PRECAUTIONS, SIDE-EFFECTS

- Should any incidents or side-effects occur during the use of LAPAROX<sup>®</sup>, which are not described in chapter 5.1.3, these must be reported to the manufacturer Linde Gas Schweiz.
- If you notice side-effects that are already described in chapter 5.1.3 you can also report them to the manufacturer Linde Gas Schweiz AG.
- If a serious incident occurs in relation to LAPAROX<sup>®</sup>, this serious incident must be reported immediately to the manufacturer and to the competent authority of the Member State in which the user and/or the patient is established.

## 5.1 Insufflation Gas

### 5.1.1 Contraindications

LAPAROX<sup>®</sup> must not be used during surgical hysteroscopic procedures because of the increased risk of gas embolism.

Procedure specific contraindications must be taken into consideration before applying CO<sub>2</sub>.

### 5.1.2 Warnings and precautions

- Before using LAPAROX<sup>®</sup> as insufflation gas, the specific risks of the respective medical procedure and the patient-specific risks must be evaluated and considered.
- During CO<sub>2</sub> insufflation, patients must be appropriately monitored for vital functions, hypercapnia, and adverse effects (specifically embolism) caused by CO<sub>2</sub> retention.
- Hypercapnia should be prevented by appropriate anesthetic and ventilation management in addition to adequate pressure control. In the presence of significant hypercapnia, appropriate medical treatment must be initiated.
- Pressure caused by insufflation can lead to mechanical tissue damage and excessive CO<sub>2</sub> absorption, which in turn may have severe systemic consequences.
- Always use the lowest-pressure pneumoperitoneum, as this may contribute to less analgesic requirement, shorter hospitalization, decreased patients' pain perception and higher satisfaction about surgery as well as possibly limit nausea and vomiting the day after laparoscopy.

- Clinical presentation of CO<sub>2</sub> embolism ranges from asymptomatic to neurologic injury and, cardiovascular collapse, which is dependent on the rate and volume of carbon dioxide entrapment and the patient's condition.
  - Patient monitoring is essential as also a low-grade embolism may develop into more serious complications.
  - The healthcare team must coordinate to prevent CO<sub>2</sub> embolism by preventing gas entry into circulation during laparoscopic surgery.
  - In case of signs for CO<sub>2</sub> embolism appropriate action must be taken according to clinical guidance.
- 
- Laparoscopic CO<sub>2</sub> insufflation can have significant haemodynamic and ventilatory effects. In patients with known cardiovascular disease special risk consideration must be made before undergoing procedure.

### 5.1.3 Side-effects

- CO<sub>2</sub> retention
- CO<sub>2</sub> embolism. In case of sudden onset of unusual arrhythmia, systolic and/or diastolic murmurs, acute cardiovascular depression or a sudden drop in end-expiratory CO<sub>2</sub> concentration, the presence of a rare gas embolism must be considered.
- Emphysema i.e., skin emphysema or scrotal emphysema

- Pneumomediastinum or Pneumothorax
- Intraoperative or postoperative pain and abdominal discomfort
- Systemic consequences due to intra-abdominal pressure increase and excessive reabsorption of CO<sub>2</sub>, which can lead to hypercapnia as well as alterations in metabolism, vasovagal response, immunological response, cerebrovascular response, vascular resistance, and hemodynamic properties.
- Abdominal or shoulder tip pain
- Nausea, vomiting
- Bloating
- Decrease in cardiac output due to decrease in preload and increase in afterload
- Abdominal insufflation impacts pulmonary function
- Secondary tissue/organ/nerve damage or abdominal compartment syndrome due to intra-abdominal pressure increase
- Risk of intraoperative hypothermia

#### 5.1.4 Interactions

Interaction of LAPAROX® with other medication has not been reported.

#### 5.1.5 Pregnancy and lactation

Current available scientific data does not allow a general recommendation on the risk benefit profile during pregnancy and breast-feeding. This is mainly not related to the medical device (CO<sub>2</sub>) itself as by the procedure. The decision for the use must

be taken by the physician under strict consideration of the risk benefit profile of the individual patient.

### 5.1.6 Roadworthiness and operation of machines

Precautions related to the procedure and anesthetics must be taken in account. No special precautions are necessary related to LAPAROX®.

## 6. PRECAUTIONS AND WARNINGS FOR TRANSPORT AND STORAGE

- The carbon dioxide is supplied in pressurised cylinders. These cylinders contain both gaseous and liquid product.
- Only store and transport gas cylinders with closed valves and with the protective device provided (e.g., protective cap).
- Protect the valve from mechanical overstressing.
- During storage and transportation, restrain cylinders with more than 3.5 litres using appropriate means (chains, hooks, etc.) to keep them in a vertical position and prevent them from falling.
- Store at ambient temperature below 50 °C and do not expose to sunlight or heat, burst plate of the cylinder may burst if heated.
- Store in a clean, well-ventilated room. Gas/vapours are heavier than air. They can accumulate in closed rooms, especially on the floor or in low-lying areas.

- Therefore, do not place cylinders in stairwells, corridors, passageways and recreation or consumption rooms.
- Empty and full cylinders must be labelled and stored separately from each other.  
When storing empty cylinders ensure that the valves are kept shut.
- The expiry date is printed on a batch label on the cylinder. Do not use the medical device after this date! The expiry date also applies to opened cylinders without restriction.  
The retest date according ADR requirements of the receptacle is indicated on the neck of the cylinder; the retest date of the test refers exclusively to the possibility of transport by road and does not refer to the expiry date of the product.

## 7. SAFE WITHDRAWAL

### 7.1 General safety instructions

- LAPAROX® may only be removed from upright cylinders. Otherwise, there is a risk of solid carbon dioxide escaping («dry ice snow»).
- Before opening the valve, pay attention to the valve, it might be cold which can lead to cold burns.
- Opening the valve abruptly is likely to lead to the liquid phase of the product and therefore introduce the risk of cryogenic burns (cold). In the event of a burn, flush with plenty of lukewarm water. Contact of solid

carbon dioxide with skin or eyes leads to frostbite (cold burns) or severe eye damage.

- Using the product at a high flow rate (> 5l/min) may cause ice on the cylinders and connections.
- Carbon dioxide is heavier than air. At high concentrations there is a danger of asphyxiation. Use only in rooms with sufficient ventilation to maintain CO<sub>2</sub> concentrations within the occupational exposure limits (OELs). Apply only by qualified personnel.
- The use of gas exhaust and/or filtering unit during minimal invasive surgery using insufflation gas is recommended.
- In the event of gas escapes, close the leaking valve, ventilate the room well and evacuate it. Never use leaking cylinders and check that emergency measures have been taken.
- Check the cylinder before use to ensure that it is in perfect condition and suitable for the intended use.
- Remove the tamper-evident seal from the cylinder; ensure that the cylinder's fitting is clean and free of foreign bodies. Open the cylinder valve slowly until a hissing sound can be heard, then close it immediately (this procedure purges any foreign bodies). Repeat the procedure 2 to 3 times.

- In the event of anomalies, e.g., in the product application, pressure indication or labelling, do not use the product and contact the manufacturer.
- It should be noted that the gas pressure in the cylinders remains constant regardless of the level of the remaining liquid (57.3 bar at 20 °C) and thus does not allow any conclusion to be drawn about the remaining quantity. As CO<sub>2</sub> is a compressed liquefied gas, the weight will decrease quickly as soon as there is no more liquid in the cylinder. Therefore, during use, only the weight of the cylinder can give an indication of the remaining liquid level.

CO<sub>2</sub> is available in pressurised cylinders, for safe withdrawal the following must be observed:

- Before each use, the compressed gas cylinder must be checked gravimetrically for the amount of CO<sub>2</sub> present.

### **Safe removal of individual cylinder**

The filling quantity (kg) results from the product of the filling factor (0.75) and the specific cylinder volume, stamped on the cylinder shoulder.

Example 2-litre cylinder:  $2 \text{ l} \times 0.75 \text{ kg/l} = 1.5 \text{ kg CO}_2$

Actual weight minus tare weight results in actual quantity of CO<sub>2</sub> available.

### **Safe removal of bundles**

Before use, check the bundle for residual pressure.

Bundle à 12 single cylinders Quantity approx. 450 kg.

Ensure that always a reserve cylinder / bundle is available.

- Do not lift cylinders by their valves.
- Never squeeze a cylinder into a holder, into which it is difficult to fit.
- Misuse of the pressure containers, filling by the user or by third parties and decanting into other cylinders are not permitted. Do not transfer compressed gas from one cylinder to another.
- The cylinders must not be completely emptied; they must be returned with a residual pressure of at least 3 bar. This residual pressure is necessary to protect the gas cylinder from contamination and liquid ingress or to prevent corrosion due to moisture.

## **7.2 Operating CO<sub>2</sub> cylinders with a pressure reducer**

- Carbon dioxide is not sterile and not free of particles. For use during laparoscopic surgery, it is mandatory to use a single-use 0.22 µm bacteriological filter that is compatible with CO<sub>2</sub> (non-sterile gas) and designed for pressures of at least 20 mmHg.
- CO<sub>2</sub> may only be used after vaporisation (conversion to the gaseous state). Vaporisation is a physical process that takes place automatically in the cylinder or bundle.
- Check the output connection of the cylinders for cleanliness before connecting the pressure reducer; clean any dirty connections with an oil- and grease-free cloth. Keep the interface between the cylinder

- and the pressure reducer clean. Check the condition of the seals.
- Only open cylinder with a pressure reducer connected. Before opening the cylinder valve, check the connected fittings (e.g., pressure reducer, flowmeter) for their closed condition. The pressure reducer must be relieved.  
Open the cylinder valve slowly turning anticlockwise, never with force and as far as it will go. Tighten the pressure reducer/flowmeter preferably by hand to avoid damaging the seals.
  - Never stand in front of the valve but always position yourself on the side opposite the pressure reducer, behind the cylinder and at some distance from it. Never expose the patient directly to the gas flow.
  - At low room temperatures or if intensive use causes the cylinder to cool down, the flow may drop or stop due to insufficient pressure in the cylinder.  
In case of heavy icing at the pressure reducer, interrupt the withdrawal and continue to use the cylinder only after it has completely thawed.
  - Close the cylinder valves after use, let the pressure of the pressure reducer drop by holding the flow meter open, close the flow meter and then loosen the adjusting screw of the pressure reducer (not with integrated pressure reducers).

### 7.3 Fittings and application devices

- Please verify that your cylinder is intended to provide gaseous CO<sub>2</sub>.
- CO<sub>2</sub> supplied in its cylinder must be connected to one or more other medical devices (usually a pressure reducer and an insufflator) to fulfil its intended purpose. The choice of suitable device depends on the application and is the responsibility of the user
- Before use, read the manufacturer's instructions supplied with the equipment or device associated with the medical CO<sub>2</sub> and follow the recommendations of the physician or surgeon. In particular, check the suitability and condition of the equipment.
- The fittings and application devices used must be compatible and approved for the intended use. Use standardized fittings or hoses specifically designed for medical CO<sub>2</sub>. Use a specific connection of type DIN 477 No. 6 (W21,8 × 1/14") or DIN EN 407 as described in chapter 10.
- Connect the cylinder to the associated device:
  - Screw fitting: align the fittings on the regulator and the cylinder.
  - Clamp fittings: Match the positioning pins to the specific holes.
- Screw on the equipment by hand until it cannot be tightened any further (in some cases, using a spanner can damage the seal).
  - Before opening the valve, check that the connections have been fitted correctly.

- Open the cylinder valve slowly, without forcing it or opening it fully.
- Always close the cylinder valve after use
- Do not disconnect the connector from the cylinder without first purging the residual compressed gas.
- Before returning for refilling, remove all accessories, hoses, etc. that were not already connected on delivery.
- Gas leak: close the valve and bleed the connection system. If the leak persists, put the cylinder outside and release the gas without attempting to patch or repair the valve. Return the defective cylinder to the manufacturer.

## **8. CLEANING AND DISINFECTION**

- Before cleaning and disinfection, close the application device and, if necessary, release the pressure from the connected fittings.
- If exterior cleaning is required, please use only clean cloth. The cloth can be dry or moistened with clean water. We recommend wiping disinfection of the outer surfaces.
- For disinfection wipe (not spray) cylinders with 70 % solution of Isopropanol (IPA) or 70 – 75 % Ethanol in water or as an alternative wipe (not spray) cylinders with 0.5 – 1.5 % solution of H<sub>2</sub>O<sub>2</sub> (hydrogen peroxide) in water. If other disinfection solutions are used, check that they are compatible with brass, plastic

materials of the components (including the stickers) and the medical gas.

When the application unit is used in a hospital, the requirements and practices of the hospital hygiene plan must be observed accordingly.

- Do not immerse the valves in water or other liquids. Do not bring any liquids into the application connections.
- Only hygienically perfect and clean cylinders (without gross contamination) may be returned.

## 9. MAINTENANCE, SERVICING AND DISPOSAL

→ Error - Cause - Remedy

Error	Cause	Remedy
Leakage from e. g. → Valve/cylinder connection → Filling connection → Pressure gauge on pressure reducer → Bundle screw connection		Close the cylinder valve/ bundle valve, move the containers to a ventilated area and call your Linde Gas Schweiz Service.
No gas flow, although the pressure gauge shows that the cylinder is not empty.	1. Cylinder valve is closed. 2. Malfunction	1. Open the valve by turning it anticlockwise. 2. Call the Linde Gas Schweiz Service.

- No modifications may be made to the cylinder.
- Do not carry out repairs on a defective valve. Repair and maintenance may only be carried out by Linde Gas Schweiz authorised and qualified personnel.
- Unauthorised maintenance or repair will inevitably lead to the exclusion of liability.
- Linde Gas Schweiz cylinders are always to be returned to Linde Gas Schweiz and not to dispose of them yourself.

## 10. FURTHER INFORMATION

- Please feel free to contact Linde Gas Schweiz with any questions or anomalies.
- Carbon Dioxide as a medical device is classified as class IIa according to Annex VIII of the European Medical Device Regulation (EU) 2017/745.
- Carbon dioxide is a very stable, non-flammable and inert, colourless and odourless gas that is heavier than air and has the following physicochemical properties:
  - Molar mass: 44,010 g/mol
  - Sublimation point at 1.013 bar: 194.65 K (-78,5 °C)
  - Vapour pressure at 20 °C: 57.3 bar
  - 1 kg of medical gas contains as active ingredient component: carbon dioxide, at least 995 g.  
The medical device contains no other ingredients.
- Further product and safety relevant information is available in the corresponding product and safety data sheet:  
[Sicherheits- und Produktdatenblätter | Linde Gas Schweiz](#)
- Stay away from MR scanners. The cylinders are not compatible with magnetic fields (MRI).
- The cylinder is equipped with a valve, with rupture disc.

→ Properties of the packaging

Material number	3750172	3750173	3750173V09	3750112	37501811	37505883
Cylinder	Steel	Steel	Steel	Steel	Steel/ Aluminium	Steel
Cylinder Volume (Liter)	2	3,5	3,5	10	40	600
Valve standard	DIN 477 No. 6 (W21,8 x 1/14")	DIN 477 No. 6 (W21,8 x 1/14")	DIN EN ISO 407	DIN 477 No. 6 (W21,8 x 1/14")	DIN 477 No. 6 (W21,8 x 1/14")	DIN 477 No. 6 (W21,8 x 1/14")
Valve	No RPV clockwise	No RPV clockwise	RPV clockwise	No RPV clockwise	RPV clockwise	RPV clockwise
Weight (kg)	5	7,5	7,5	26	76	1500
Dimensions with cap h x Ø (mm)	415 x 102	650 x 100	650 x 100	930 x 140	1610 x 205	Bundle Height: 196 cm Length: 100 cm Width: 75 cm

RPV - Residual Pressure Valve      Colour indication:      Cylinder shoulder dusty grey (RAL 7037)  
Cylinder body pure white (RAL 9010)

## 11. LABELLING

Manufacturer

Medical Device

EU representative

Conformity mark of the medical device

0123

CE marking of the devices (incl. identification number of the notified body responsible for the conformity assessment procedures)

Catalogue number, Article number

Unique Device Identifier

Batch number

Use-by date, Expiry date

Caution, warning notice

Electronic instruction for use  
Read the electronic instructions for use before  
use at

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For printed instructions for use,  
call 0844 800 300.

Material Number	Material Name	UDI-DI
3750172	LAPAROX® CO <sub>2</sub> MED 2 L	111756893460
3750173	LAPAROX® CO <sub>2</sub> MED 3,5 L	111756894026
3750173V09	LAPAROX® CO <sub>2</sub> MED 3,5 L PI	111756895716
3750112	LAPAROX® CO <sub>2</sub> MED 10 L	111756896379
37501811	LAPAROX® CO <sub>2</sub> MED 40 L	111756898635
37505883	LAPAROX® CO <sub>2</sub> MED 600 L	111756899201

Linde Gas Schweiz AG  
Industriepark 10  
CH-6252 Dagmersellen  
Telefon 0844 800 300, Fax 0844 800 301  
contact.lg.ch@linde.com, www.linde.ch

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Linde GmbH, Gases Division