

Recently viewed patients

| | Name | Gender | Age | Patient ID | Device Model | Device SN | Working Mode | Patient Group | Download Time | Data Size |
|---|-------|--------|-----|------------|--------------|-------------|--------------|---------------|---------------------|-----------|
| 1 | Demo2 | Male | 51 | | G2S A20 | ES418C03344 | AutoCPAP | Default | 2021-06-24 17:14:18 | 4276 k |
| 2 | Demo1 | Male | 51 | | Y-30T | Y2C16A01050 | S/T | Default | 2021-06-24 17:09:52 | 17645 k |

Choose the *.USR file that you can get it from your SD card

Data download ×

Data

File path

Device SN Device Model

Link of

Please enter the name, patient ID or device serial number to

| ▲ | Name | G.. | Age | Patie... | Devic... | Devic... | Worki... | Patie... | Download Time | Data ... |
|---|------|-----|-----|----------|----------|----------|----------|----------|---------------|----------|
| | | | | | | | | | | |

Download

Date Range of Summary Data

Detailed Graph Data

1. You can build a new patient profile
2. If you have already built one, then choose the right one to contain the data
3. You can also choose to Link no patient

Patient Profile

Patient Information

Clinical Information

1. Type in the basic information

Basic Information

* Name

Patient ID

Gender

Male

Female

* Date of Birth

1970/01/01

Age

51

Height (cm)

Weight (kg)

BMI

Patient Group

Default

Contact Information

Address

District

City

Province

Postcode

Country

Phone Number

E-mail

Doctor Information

Doctor

Department

Hospital/Clinic

Insurance Information

Insurance Carrier

Insurance Membership

Start date

2021/06/24

* Required Field

2.

Confirm

Cancel






Recently viewed patients

Download and View: you can view the data after data downloading

If choose “Download and Generate Report”, the Report window will pop up, choose the report type you want and get the report.

Report

Report Generation

 Usage Compliance  Statistical Data  Summary Data  Detailed Data  All Available Data

Start date 2020/04/08

End date 2020/04/08

Last 1 day

Generate

Previous Report

File Name Report Generatio...

View

Recently viewed patients


| | Name | Gender | Age | Patient ID | Device Model | Device SN | Working Mode | Patient Group | Download Time | Data Size |
|---|-------|--------|-----|------------|--------------|-------------|--------------|---------------|---------------------|-----------|
| 1 | Demo3 | Male | 51 | | G3 A20 | A3119601102 | AutoCPAP | Default | 2021-06-24 17:18:06 | 18594 k |
| 2 | Demo2 | Male | 51 | | G2S A20 | ES418C03344 | AutoCPAP | Default | 2021-06-24 17:14:18 | 4276 k |
| 3 | Demo1 | Male | 51 | | Y-30T | Y2C16A01050 | S/T | Default | 2021-06-24 17:09:52 | 17645 k |

Data browsing

Start date 2020/04/08 ▾

End date 2020/04/08 ▾ ▴

Last 1 day

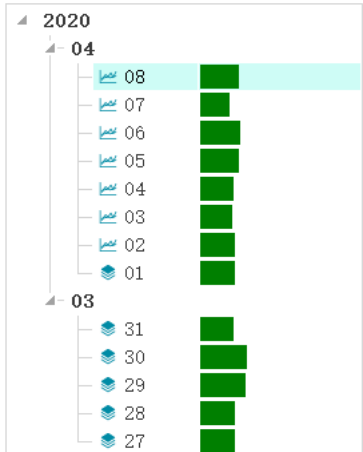

Recent Usage

Device Model G3 A20


```
Device SN      A3119601102
```

Compliance of last 30 days



2020/03/10 2020/04/08


[Daily List](#)
[Current Settings](#)
[Statistics](#)
[Summary Graphs](#)
[Detailed Graphs](#)
[Report Preview](#)

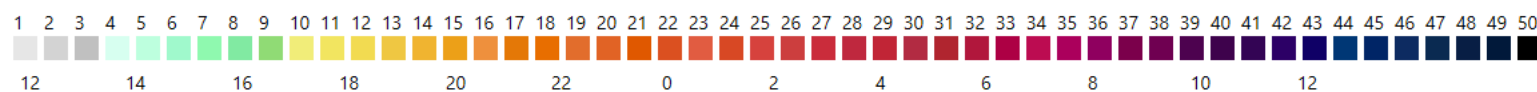

| ▲ | Date | | | | | | | | | | | Used Time | Mean P | P95 | AHI | Leak |
|----|------------|--|--|--|--|--|--|--|--|--|--|-----------|--------|-----|-----|------|
| 1 | 2020/04/08 | | | | | | | | | | | 06:00 | 7.0 | 8.5 | 0.3 | 1.1 |
| 2 | 2020/04/07 | | | | | | | | | | | 04:29 | 7.0 | 8.5 | 1.1 | 1.1 |
| 3 | 2020/04/06 | | | | | | | | | | | 06:08 | 7.5 | 8.5 | 0.8 | 1.0 |
| 4 | 2020/04/05 | | | | | | | | | | | 05:58 | 7.5 | 8.5 | 0.3 | 2.6 |
| 5 | 2020/04/04 | | | | | | | | | | | 05:07 | 7.5 | 8.5 | 0.2 | 1.8 |
| 6 | 2020/04/03 | | | | | | | | | | | 04:57 | 7.5 | 8.5 | 1.4 | 0.3 |
| 7 | 2020/04/02 | | | | | | | | | | | 05:20 | 7.5 | 9.5 | 1.9 | 0.7 |
| 8 | 2020/04/01 | | | | | | | | | | | 05:24 | 7.0 | 8.0 | 1.3 | 0.4 |
| 9 | 2020/03/31 | | | | | | | | | | | 05:04 | 7.0 | 8.5 | 0.8 | 1.1 |
| 10 | 2020/03/30 | | | | | | | | | | | 07:13 | 7.0 | 8.5 | 1.4 | 0.6 |
| 11 | 2020/03/29 | | | | | | | | | | | 07:02 | 7.0 | 8.5 | 0.6 | 2.6 |
| 12 | 2020/03/28 | | | | | | | | | | | 05:17 | 6.5 | 7.5 | 0.0 | 0.9 |
| 13 | 2020/03/27 | | | | | | | | | | | 05:18 | 7.0 | 8.5 | 1.3 | 2.2 |

2. Click here to save the report

Data browsing

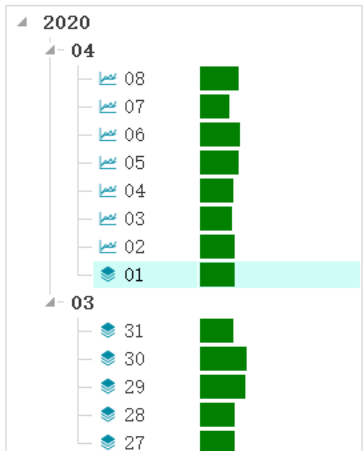Start date: 2020/04/01
End date: 2020/04/01
Last: Customized

Recent Usage

Device Model: G3 A20
Device SN: A3119601102
Compliance of last 30 days: 43%
2020/03/10 2020/04/08Daily List Current Settings Statistics Summary Graphs Detailed Graphs **Report Preview**Patient Name: Demo3
Patient ID:
Report prepared by:
Report generated on: 2021/06/24

TREATMENT REPORT

Patient Profile

Basic Information

Name: Demo3
Gender: Male Age: 51 Date of Birth: 1970/01/01
Height: 0.0cm Weight: 0.0kg BMI: 0.0 Patient ID:

Contact Information

Address:
Phone Number: E-mail:

Doctor

Clinic / Hospital:
Doctor: Department:

Insurance



Insurance Carrier:
Membership Number: Member Since: 2021/06/24

Device Information

Device Type: Auto CPAP Device Model: G3 A20
Device SN: A3119601102 Firmware Version: nP3-

Report Type

All Available Data

Report Type

All Available Data

Usage Compliance

Statistical Data



Summary Data

Detailed Data

All Available Data

1. Choose the report type

Summary

1. Click on Data Download 
Data Download
2. Choose the *.USR data file obtained from your SD card
3. You can firstly build a new patient profile, or if you have already built one, you can choose the right one directly. You can also choose to Link no patient
4. Type in the basic information and confirm
5. Download Only: it helps you download the data quickly;
Download and View: you can view the data after data downloading;
Download and Generate Report: you can get the appropriate report directly.
6. In the Report Preview page: Save the report by clicking the save icon , also the report type can be selected at the right side.