


Instrucciones

Instalación del certificado en Edge (versión 85+)


AirView y ResMed Cloud Connect requieren un certificado para que puedan trabajar a la vez. Solo tendrá que completar este paso una vez. La instalación tardará solo unos minutos.

Instrucciones de impresión

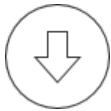


1. Descargue el certificado


2. En el navegador, haga clic en **server.crt** en la esquina inferior izquierda.

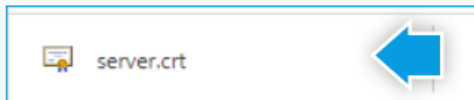

3. Haga clic en **Abrir**.

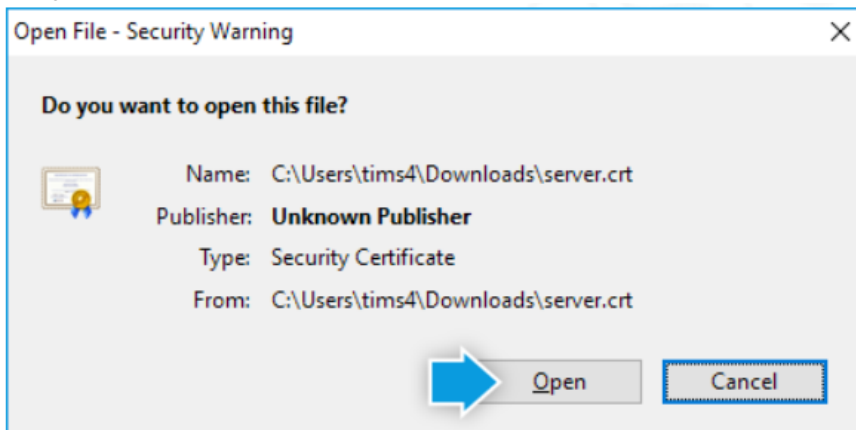


4. Haga clic en **Instalar certificado**.

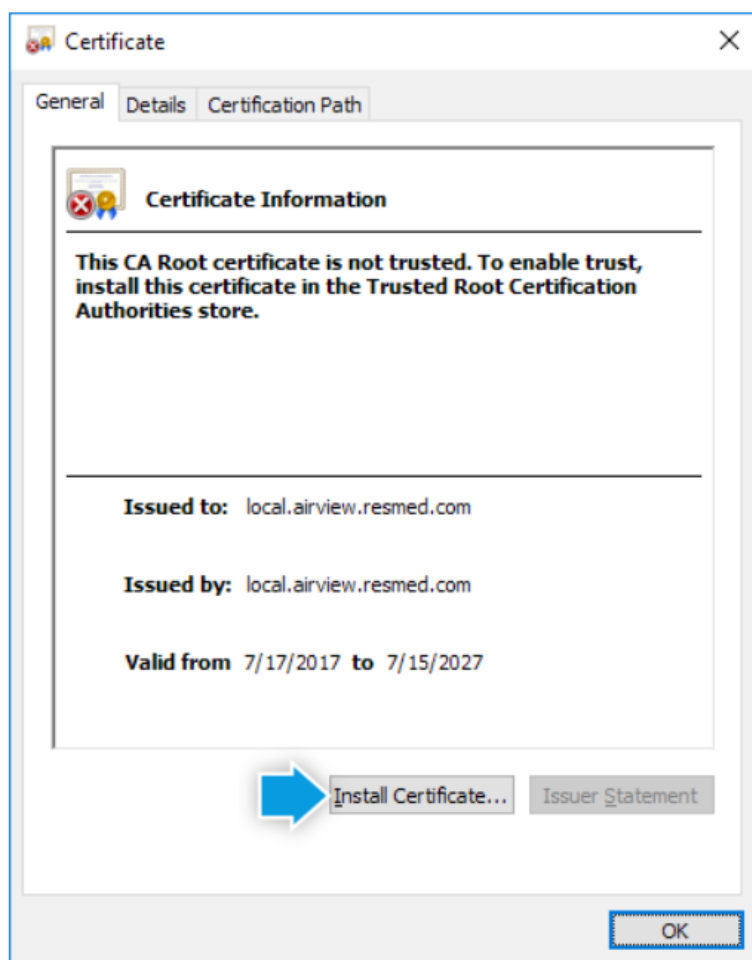

5. En la sección **Ubicación de almacenamiento**, seleccione **Equipo local**.

6. Haga clic en **Siguiente**.

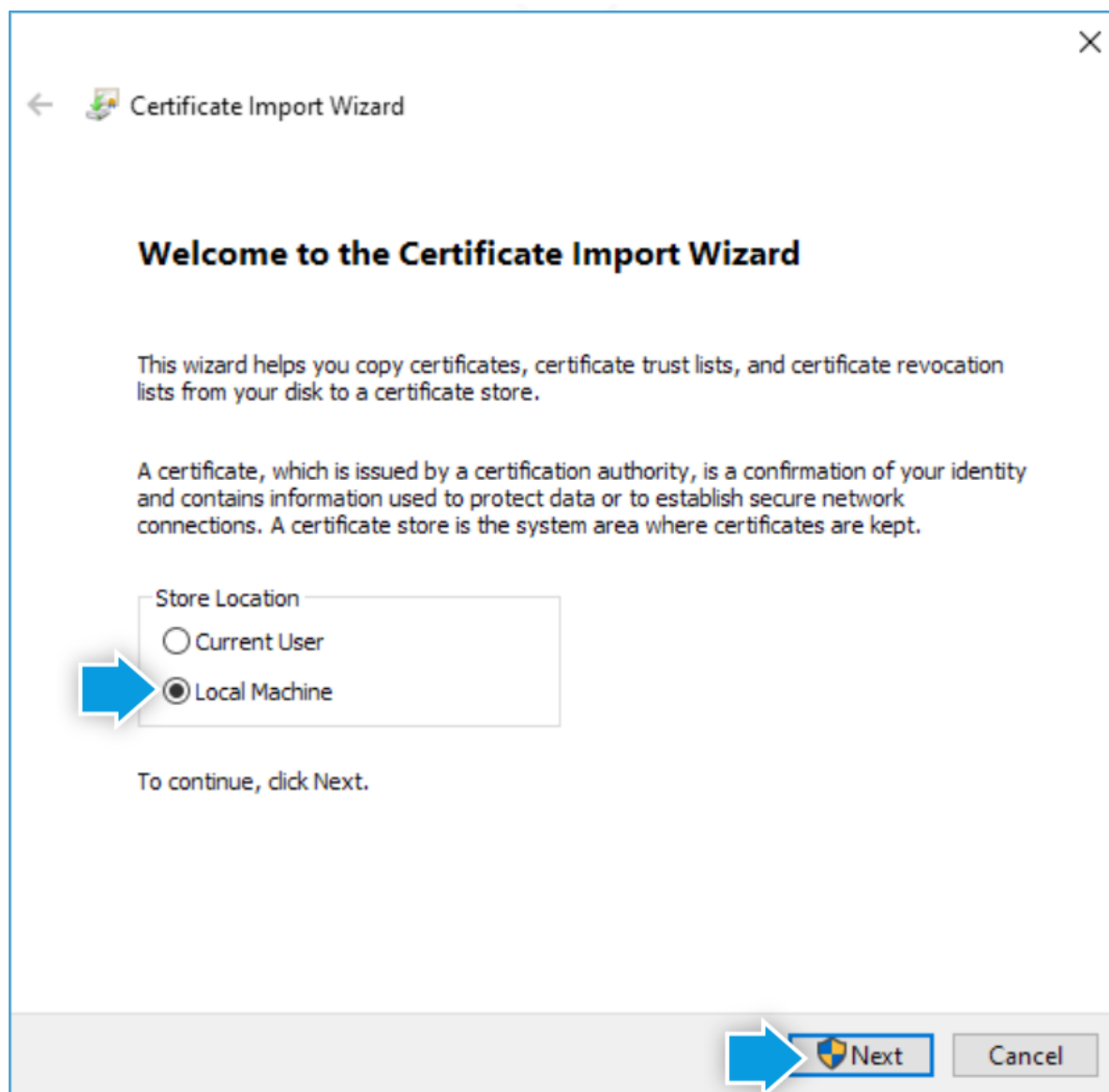

7. Seleccione **Colocar todos los certificados en el siguiente almacenamiento**.

8. Haga clic en **Examinar**.



Certificate Import Wizard

Certificate Store

Certificate stores are system areas where certificates are kept.

Windows can automatically select a certificate store, or you can specify a location for the certificate.

☐ Automatically select the certificate store based on the type of certificate



☒ Place all certificates in the following store

Certificate store:



Browse...

Next

Cancel

9. Seleccione **Entidades de certificación raíz de confianza**.

10. Haga clic en **Aceptar**.



Select Certificate Store

Select the certificate store you want to use.

- Personal
- Trusted Root Certification Authorities
- Enterprise Trust
- Intermediate Certification Authorities
- Trusted Publishers
- Untrusted Certificates

☐ Show physical stores

OK Cancel

certificates are kept.

store, or you can specify a location for

based on the type of certificate


re

Browse...


Next

Cancel

11. Haga clic en **Siguiente**.

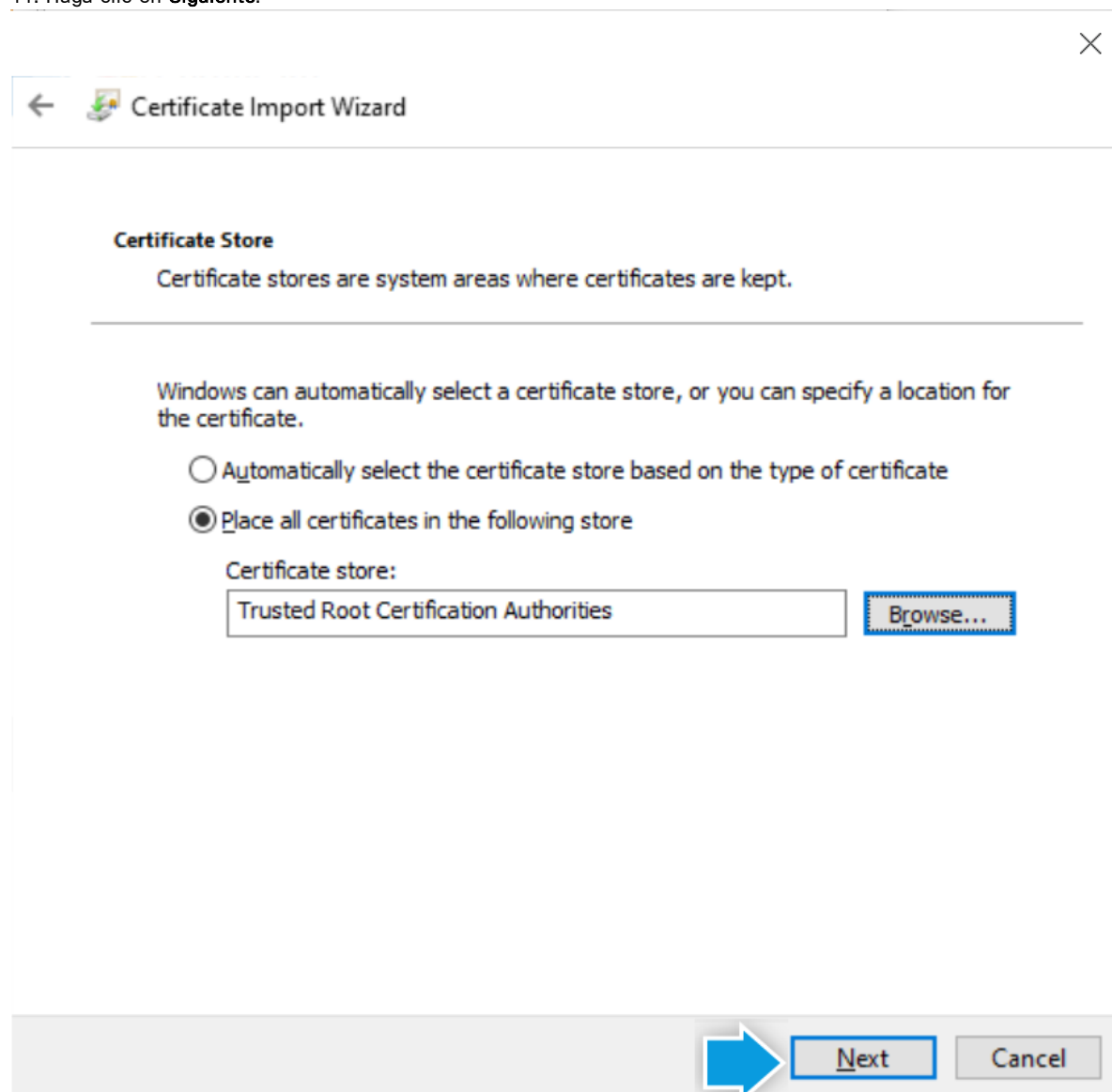

12. Haga clic en **Finalizar**.

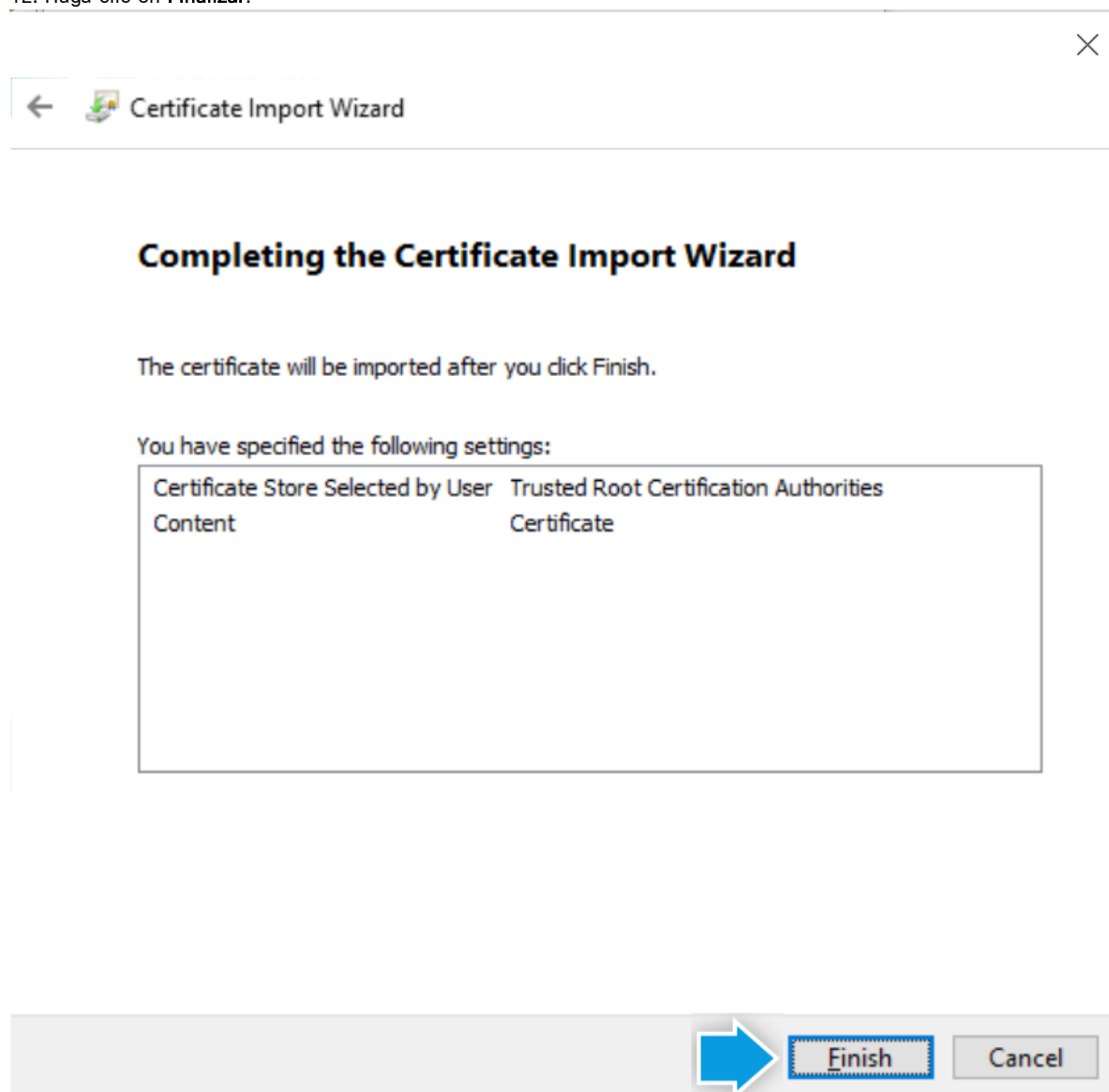

13. En AirView, seleccione **He instalado el certificado según las instrucciones** y haga clic en **Siguiente**.